

eylys

EYE CARE KIT

Heated Eye Mask + Massage Roller

Consistent, gentle warmth for a simple everyday eye-care routine.

MODEL ECKW



DESIGNED BY A CORNEA SPECIALIST

An Oculab eye-care innovation

AVAILABLE THROUGH JK MEDICAL IN LATIN AMERICA

A More Consistent Warm-Eye Routine

Designed to replace the inconvenience and rapid cooling of traditional warm compresses

THE IDEA BEHIND EYLYS

Cornea specialist Dr. Hemal Kenia developed the Eylys Eye Care Kit after observing how difficult it can be for people to maintain a practical warm-eye routine. Following focused product development, Oculab introduced a reusable system that combines a contoured warming mask with a manual eyelid massage roller.



EVERYDAY DESIGN PRIORITIES

1 3D eye-contoured fit

Raised eye-contact zones are shaped to rest around the closed eyes and support a comfortable fit.

2 Three heat modes

Blue, green, and red indicators provide low, medium, and high heat selections.

3 Five-minute session

The mask automatically switches off after five minutes for convenient, controlled use.

4 USB-C power

Connect with the supplied USB-C to USB-C cable and a compatible USB-C power source.

5 Uniform warming surface

The warming design is intended to distribute consistent heat across the eye area.

6 Portable routine

Reusable mask and manual roller support eye-care routines at home, work, or while traveling.

Designed Around the Eyes

Contoured construction, simple controls, and color-coded heat selection



COLOR-CODED HEAT MODES



BLUE

Low heat

Press the same control button to advance through the available modes.



GREEN

Medium heat

Press the same control button to advance through the available modes.



RED

High heat

Press the same control button to advance through the available modes.

A controlled five-minute warming session

The mask starts in blue mode when switched on. Select the heat level that feels comfortable, fit the contoured side gently over closed eyes, and rest during the session. The device switches off automatically after five minutes.

A Simple Eye-Care Routine

Use only as directed in the current Eylys user manual

WARMING MASK

Prepare

1

Clean and dry the face and eye area. Inspect the mask, cable, and connector before use.

Connect and select

2

Connect the supplied USB-C to USB-C cable to a compatible power source. Switch on the mask and select blue, green, or red mode according to comfort.

Position

3

Close the eyes and fasten the contoured side gently over them. Do not overtighten. Rest the head comfortably during the five-minute session.

Finish

4

The mask switches off automatically. Disconnect it after use and allow it to cool before storage or cleaning as directed.

MASSAGE ROLLER

Use gentle pressure

5

With eyes closed, use the supplied roller only as instructed. Avoid pressure on the eyeball and stop if the motion causes pain or discomfort.

Important comfort and safety note

Do not use on injured, infected, acutely inflamed, or recently operated eyes unless an eye-care professional specifically advises it. Stop use and seek professional advice for pain, persistent blurred vision, worsening redness, irritation, or symptoms that do not improve. Do not fall asleep while using the mask.

Everything Needed for the Routine

Eyls Eye Care Kit, Model ECKW



INSIDE THE BOX

1 Eyls warming eye mask

2 Manual massaging eye roller

3 USB-C to USB-C cable

4 User manual

Product overview

Model: ECKW Session: 5 minutes with automatic shut-off Heat selection: 3 modes Power connection: USB-C to USB-C Use: reusable personal eye-care device



EYE CARE KIT

Consistent warmth. Simple routine.



AVAILABLE IN LATIN AMERICA THROUGH JK MEDICAL

Availability and authorized product claims vary by country.

JK Medical, Inc.

Latin America Distributor Contact

Office: +1 (352) 324-3091

Mobile: +1 (407) 342-8575

Email: info@jkmedicalinc.com

Web: www.jkmedicalinc.com

Manufacturer

Oculab Private Limited

Mumbai, India

www.oculab.in

For personal eye-care and wellness use. This brochure is a commercial overview and does not replace the user manual or professional medical advice. Product availability, regulatory status, labeling, indications, warnings, packaging, and included accessories vary by country. Confirm the locally authorized version before sale or use. Product images are illustrative.